

# BETACHEK® C50

## Alles-in-één bloedglucosemeter

### MedM HealthApp

The collage displays several app screens:

- All Readings:** A list of glucose readings with timestamps and values (e.g., 4.8 mmol/L, 5.1 mmol/L, 5.0 mmol/L, 6.2 mmol/L).
- Summary:** A circular gauge showing 73% Normal Glucose, 19% High Glucose, and 8% Low Glucose.
- Line Graph:** A graph showing glucose levels over a period from 26 Apr 2023 to 3 May 2023.
- Details:** A detailed view of a reading: 2 May 2023, 4:15 pm, Blood Glucose 6.2 mmol/L, Before meal.
- Export:** A menu for exporting data, including options for time interval (Week, Month, 3 Months, Custom), format (Generic CSV, Excel CSV), and share type (Share, Email).

In the center, the **BETACHEK C50** device is shown with a digital display reading **90 tests**.






Download on the **App Store**

GET IT ON **Google Play**




# De Betachek C50-bloedglucosemeter koppelen met de MedM Health-app:

1. Download de MedM Health-app uit de Apple App Store of Google Play.

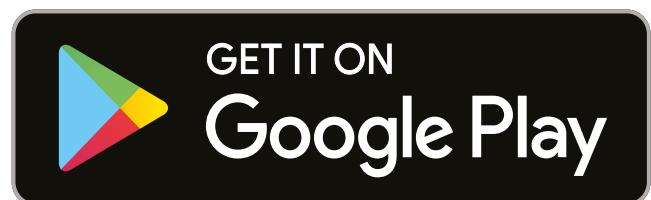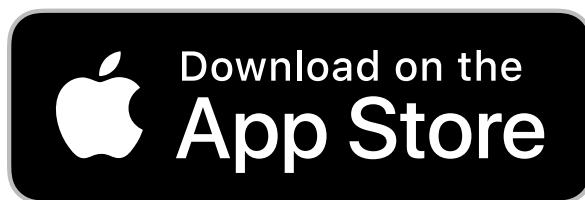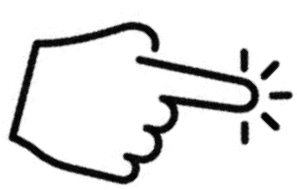

2. Open de MedM-app en druk op het menu (in de linkerbovenhoek van het scherm)

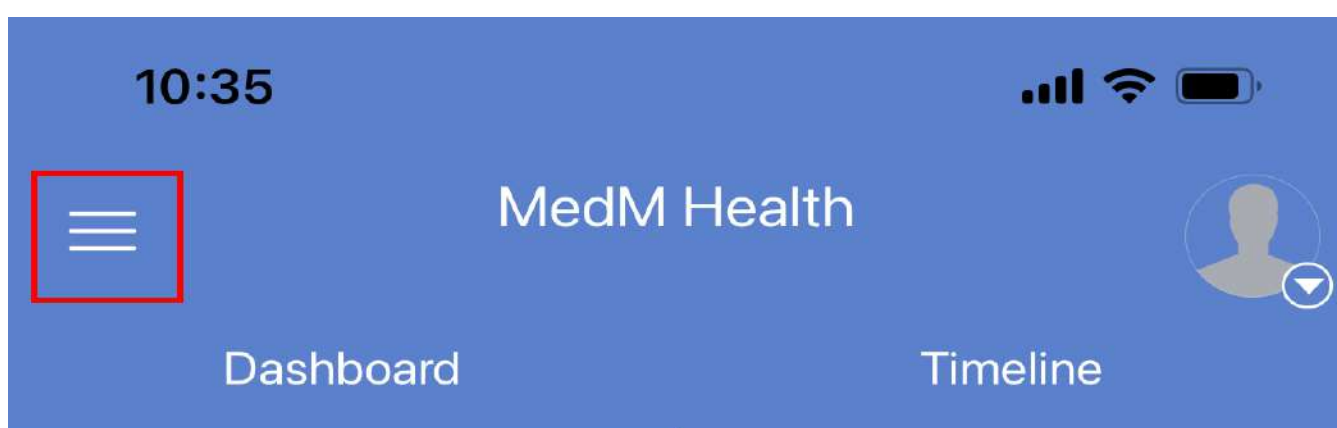


 0 steps /  
0 km /  
726 kcal /  
0% Activity Time 0s

 Blood Glucose 


 Blood Pressure  
No data yet 

 Heart Rate 


# Selecter “ My Devices”

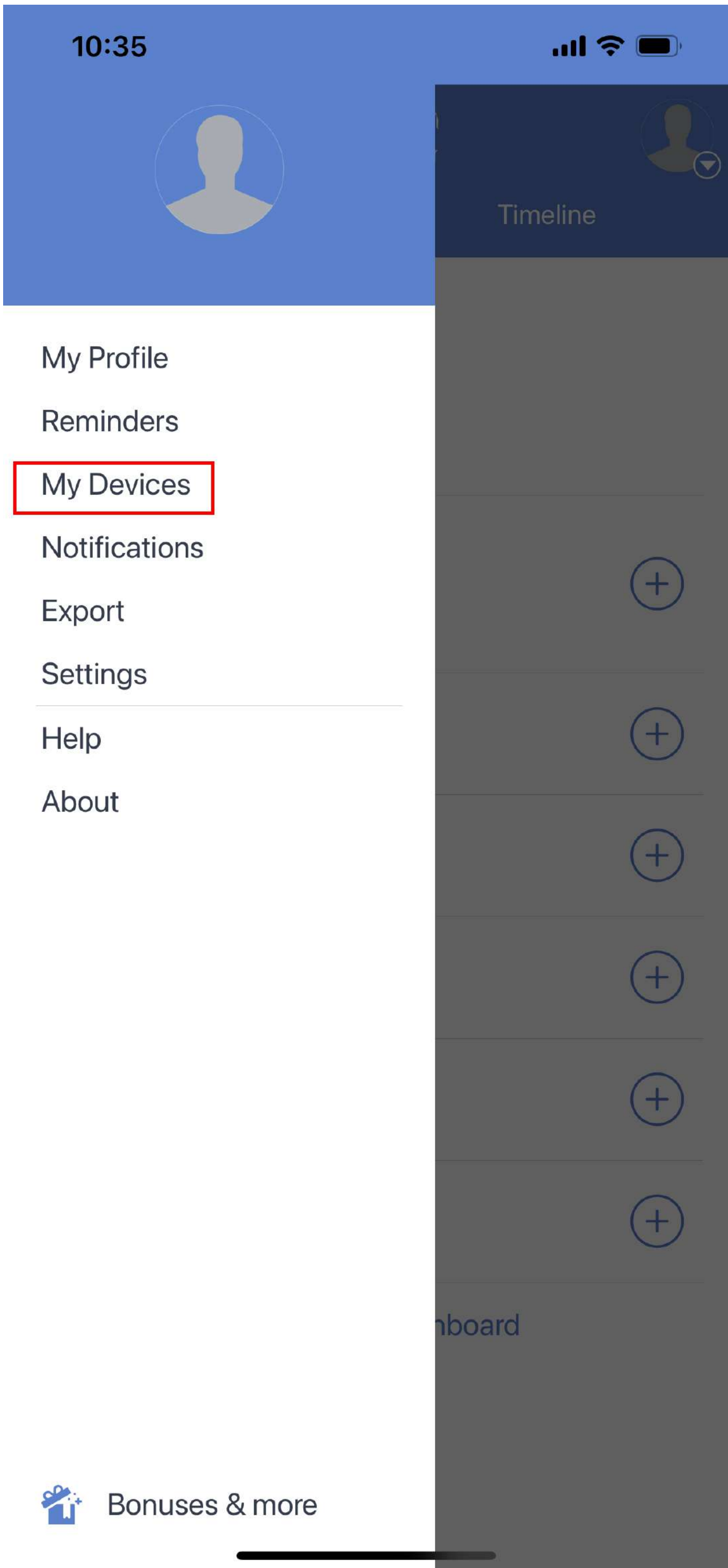

# Selecter “ Add New”

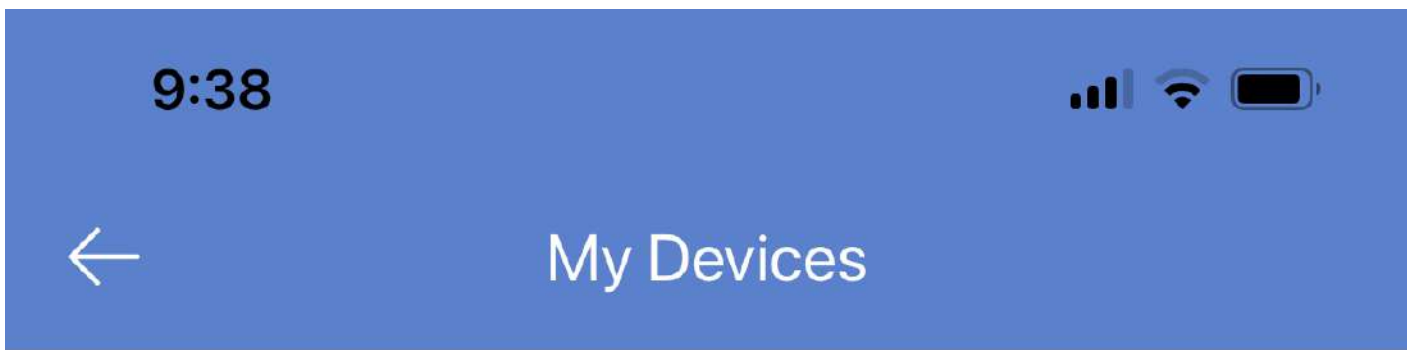

You have no devices.  
To add a device tap 'Add New' button.

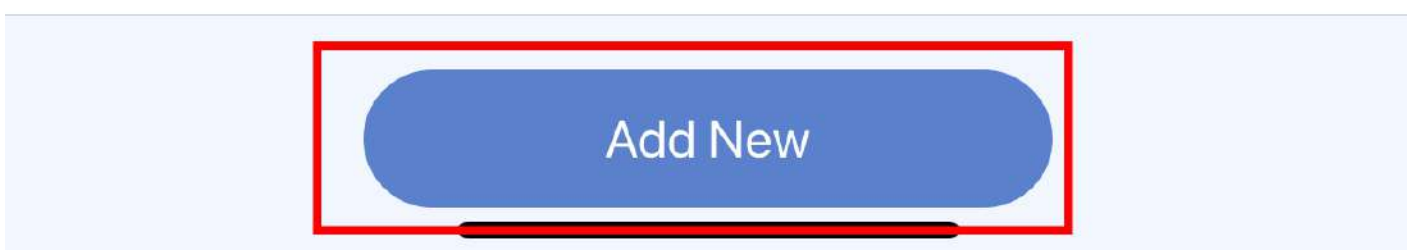

1

Houd de S- en M-knoppen ingedrukt totdat het Bluetooth-symbool verschijnt




2


De bloedglucosemeter bevindt zich nu in de koppelingsmodus



# Druk op het pijltje naast Betachek C50

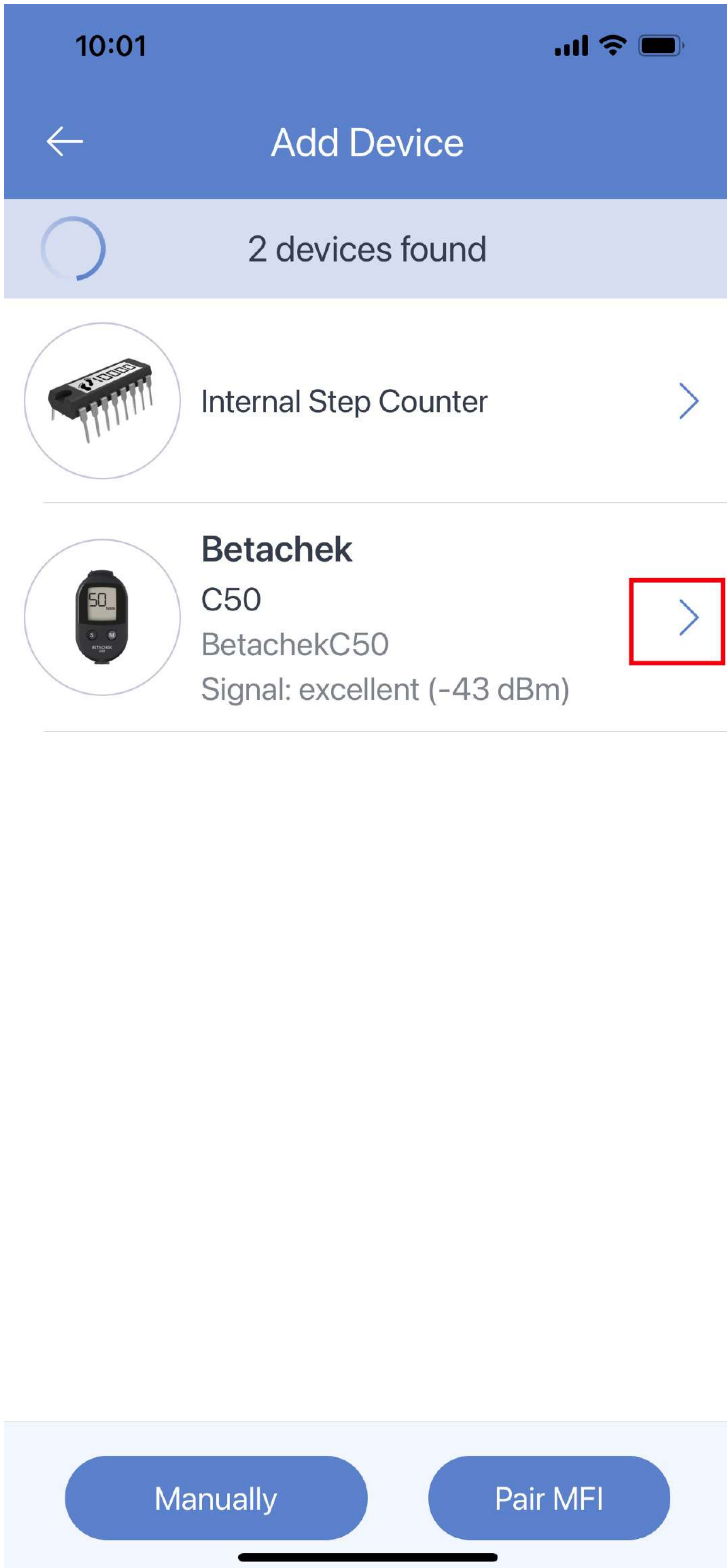

# Selecter “Add to my Devices”

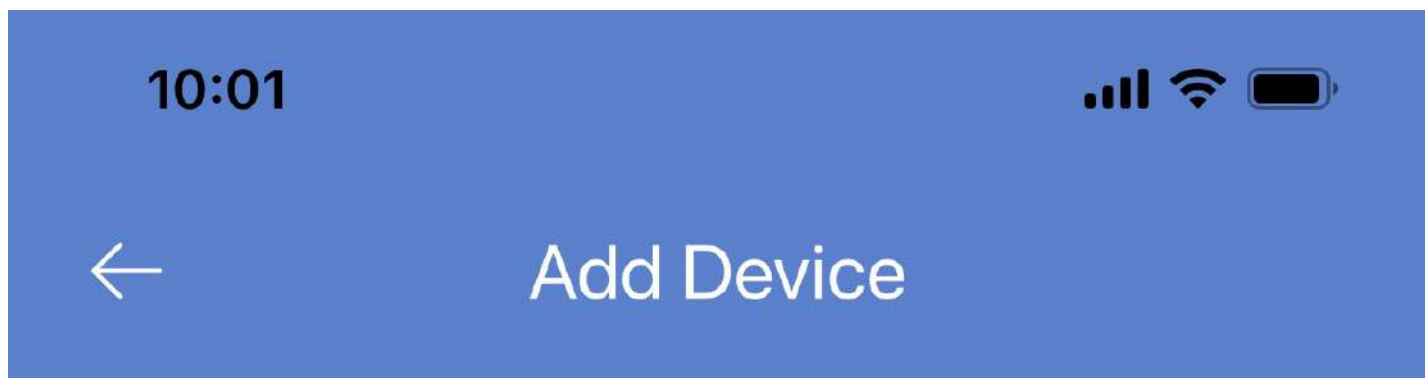

**Betachek**  
C50

Receive data automatically

App has to be in the foreground with screen unlocked.

---

Receive data in background

Collect data in background or if screen is locked.

---

**Add to My Devices**

10:01



MedM Health



Dashboard

Timeline



Blood Glucose  
No data yet



MedM Health



Betachek C50 is successfully  
paired with MedM Health.

Make a measurement to upload  
data.

Ok



Manage Dashboard



10:01



# MedM Health



Dashboard

Timeline



Blood Glucose

No data yet



Blood Pressure

No data yet



Heart Rate

No data yet



### Bluetooth Pairing Request

"BetachekC50" would like to pair with your iPhone.

Cancel

Pair



Temperature

No data yet



Weight

No data yet



Manage Dashboard