

Need Help? Email: support@betachek.com

BETACHEK® C50


All-in-One Blood Glucose Meter

MedM HealthApp

The collage displays several app screens:


- All Readings:** A list of blood glucose readings with timestamps and values (e.g., 4.8 mmol/L, 5.1 mmol/L, 5.0 mmol/L, 6.2 mmol/L).
- Summary:** A circular gauge showing 73% Normal Glucose, 19% High Glucose, and 8% Low Glucose.
- Line Graph:** A graph showing glucose levels over a period from 26 Apr 2023 to 3 May 2023.
- Details:** A detailed view of a reading: 2 May 2023, 4:15 pm, Blood Glucose 6.2 mmol/L, Before meal.
- Export:** A menu for exporting data with options for time interval (Week, Month, 3 Months, Custom), format (Generic CSV, Excel CSV), and share type (Share, Email).

In the center is the **BETACHEK C50** device, a black handheld meter with a digital display showing **90 tests** and buttons labeled **S** and **M**.

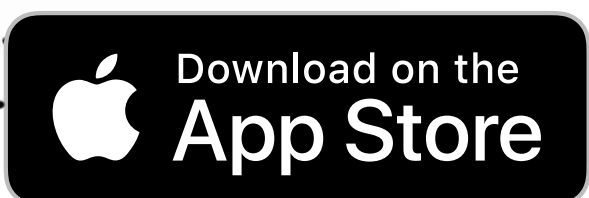

Connecting the BETACHEK C50 meter to the MedM Health-app:

1. Download the MedM Health App from the Apple App Store or Google Play.

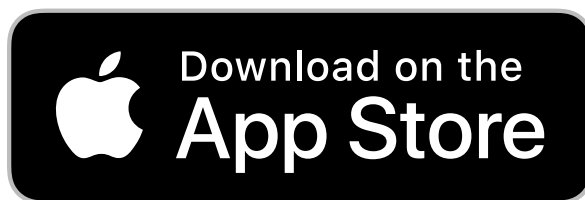

2. Open the MedM App and tap on the menu icon (three bars in the top left corner)




 0 steps /
0 km /
726 kcal /
0% Activity Time 0s

 Blood Glucose 


 Blood Pressure
No data yet 

 Heart Rate 


Select "My Devices"

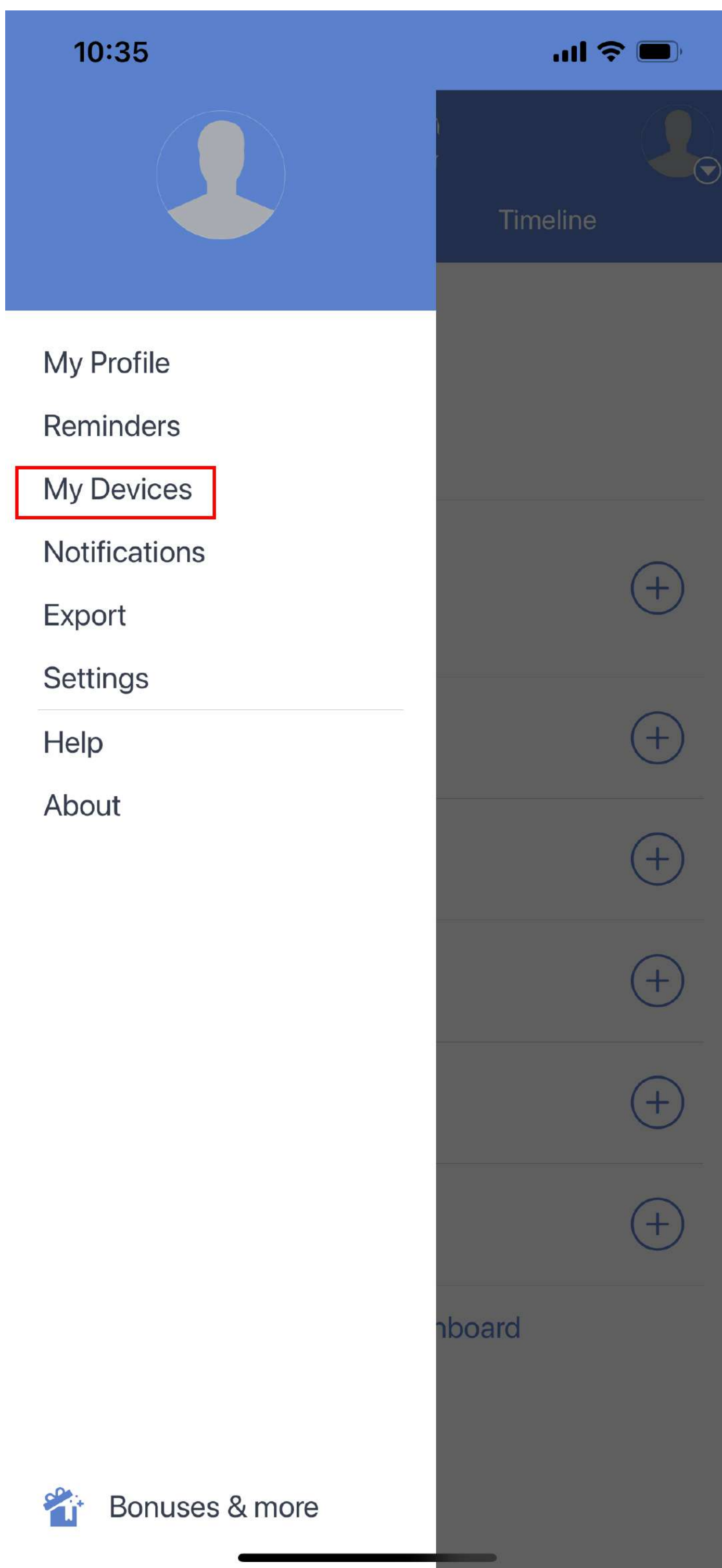

Select “ Add New”

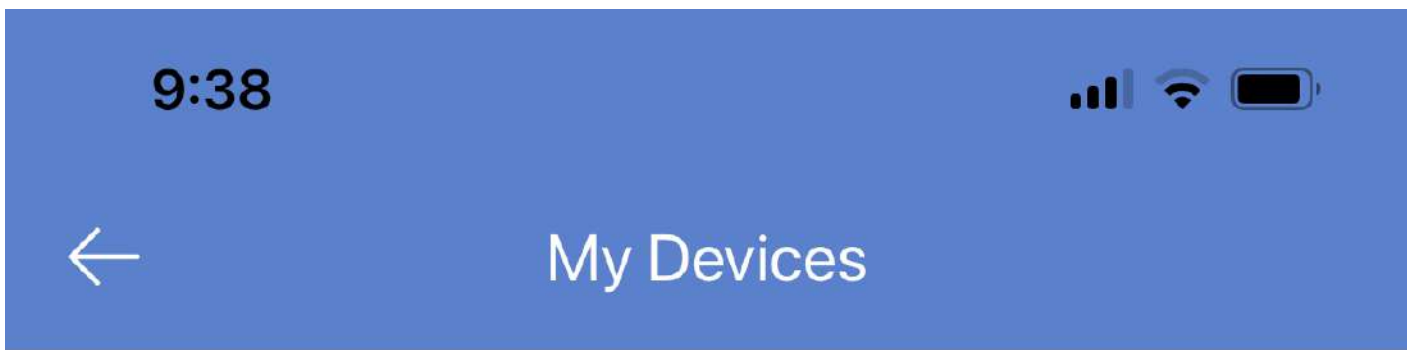

You have no devices.
To add a device tap 'Add New' button.

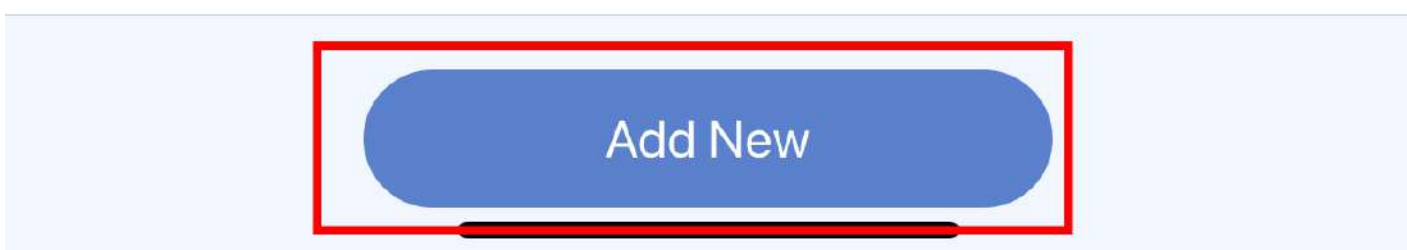

1 Hold down the S and M buttons until the Bluetooth symbol appears.



2 The bluetooth icon flashing shows the meter is ready to pair.



BETACHEK C50 will appear when the app detects the meter. Press on the arrow.

10:01 📶 📶 🔋

← Add Device


🔄 2 devices found



Internal Step Counter




Betachek
C50
BetachekC50
Signal: excellent (-43 dBm)

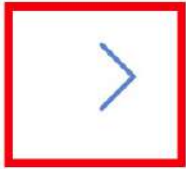

Manually

Pair MFI

Select "Add to my Devices"

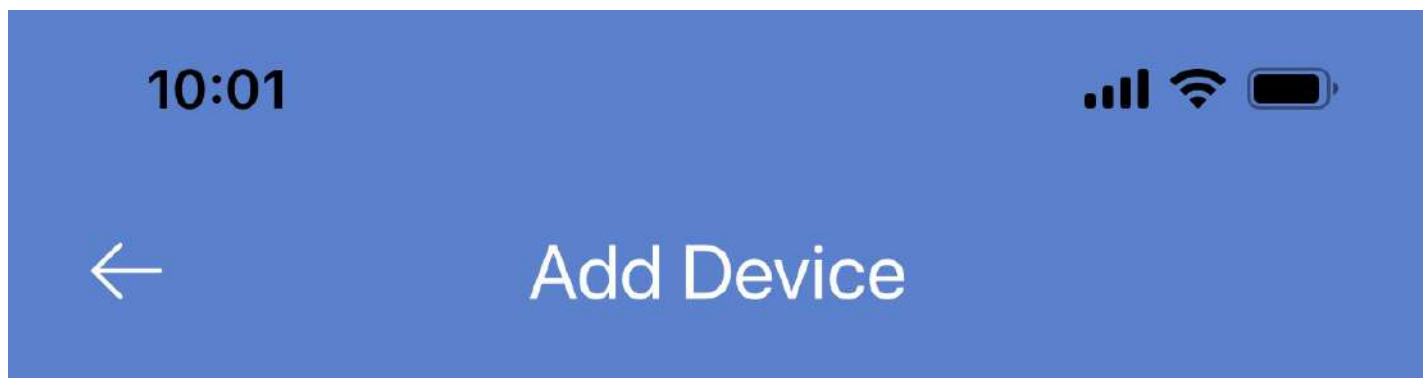

Betachek
C50

Receive data automatically

App has to be in the foreground with screen unlocked.

Receive data in background

Collect data in background or if screen is locked.

Add to My Devices

10:01



MedM Health



Dashboard

Timeline



Blood Glucose
No data yet



MedM Health



Betachek C50 is successfully
paired with MedM Health.

Make a measurement to upload
data.

Ok



Manage Dashboard

10:01



MedM Health



Dashboard

Timeline



Blood Glucose
No data yet



Blood Pressure
No data yet



Heart Rate
No data yet



Bluetooth Pairing Request
"BetachekC50" would like to pair with
your iPhone.



Cancel

Pair



Temperature
No data yet



Weight
No data yet



Manage Dashboard